
### Set Up



Start with the Meter off. Do not insert a Test Strip.

1. Hold down the **S** button until Meter beeps and full Display appears. Meter is now in Set Up mode.

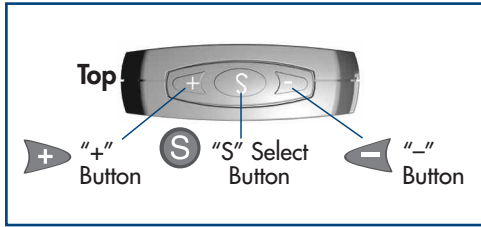


#### Setting Time:

2. The number for the hour flashes. Press the **+** button to increase or the **-** button to decrease. Press the **S** button when the correct hour is shown.
3. Repeat Step 2 for minutes.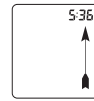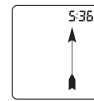

#### Setting Date:

4. The number for the month flashes. Press the **+** button to increase or the **-** button to decrease. Press the **S** button when the correct month is shown.
5. Repeat Step 4 for day.
6. Repeat Step 4 for year.

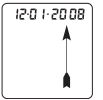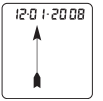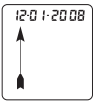For more detailed information, including a troubleshooting guide, refer to the Owner's Booklet.

### Coding Your Meter

1. Remove Code Chip from Test Strip Box. Match the number printed on the Code Chip to the number on the Test Strip Vial (plastic container).

2. Insert Code Chip into your Meter with the numbers facing up.


### Blood Glucose Testing

1. Wash hands with warm, soapy water.
2. Remove Test Strip from vial and recap vial immediately.
3. With Meter off, insert Test Strip (Contacts facing up) into Test Port. Meter turns on.

4. Confirm Meter Display and Code Chip number matches.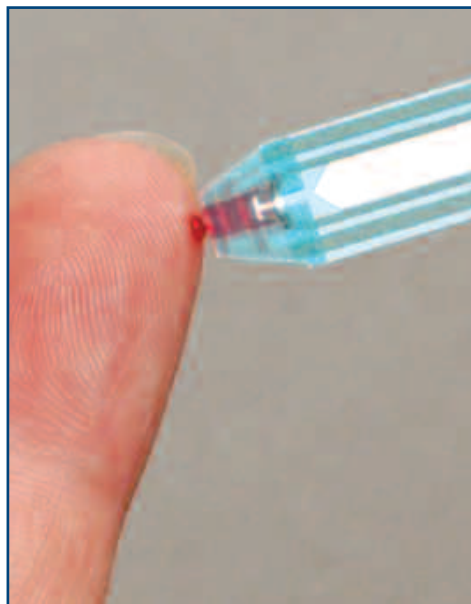

5. Lance finger. Let a blood drop form.
6. Place Sample Tip of Test Strip against blood drop and hold (see illustration). Blood is drawn into Test Strip until Meter beeps. (See Note below).
7. After countdown, glucose result is displayed with date/time. Remove Test Strip and discard. Meter turns off. Test result is stored in Meter's Memory.

**Note:** Discard Test Strip if Meter did not beep within 5 seconds after placing Tip of Test Strip against blood drop. Repeat test with new Strip and new blood sample. If problem persists, see Owner's Booklet.

### Data Management • Refer to the Owner's Booklet.

HOMEdiagnostics™

Always follow the treatment advice given to you by your doctor or diabetes healthcare professional.

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